

**GREAT LAKES ORTHOPEDIC LABS, INC.**

2362 Genesee Street, Cheektowaga, NY 14225

Telephone No. (716) 893-4116

**GREAT LAKES ORTHOPEDIC LABS, INC.**

219 Bryant Street, Buffalo, NY 14222

Telephone No. (716) 878-7307

**DIRECTIONS FOR WEARING THE  
HALLUX VALGUS SPLINT (BUNION SPLINT)**

1. Slide the big toe into the toe section of the splint. You may need to widen the forefoot section of the splint by gently pulling out the sides of the forefoot section.
2. Once the big toe is in the proper position, fasten the forefoot strap.
3. If it is difficult to slide the toe and forefoot into the splint, add some foot powder or baby powder so that it slides in easier.
4. The splint can be washed with skin soap, rinsed, and air dried.
5. The splint is designed for sleeping only—do not walk with splints on.
6. The splint will require periodic adjustments to position the big toe into its maximum corrected position.
7. Keep splints away from pets. Pets will destroy the splint if they chew on it.
8. If you have fitting problems, please call our office. If you need immediate assistance after regular office hours, please call (716) 893-4116 and leave a message on the answering machine. Your call will be returned.
9. Caution: When not wearing the device, keep it away from your pets since they are attracted to the human scent and will chew and destroy it.

Warranty: The components of this device are warranted for 90 days, after which a service charge, based on an hourly rate and materials, can be applied. The warranty does not apply to anatomical changes of the patient, misuse, and/or loss of the device or any alterations made by anyone other than Great Lakes.