

**GREAT LAKES ORTHOPEDIC LABS, INC.**  
2362 Genesee Street, Cheektowaga, NY 14225  
Telephone. No. (716) 893-4116

**GREAT LAKES ORTHOPEDIC LABS, INC.**  
219 Bryant Street, Buffalo, NY 14222  
Telephone No. (716) 878-7307

## **DIRECTIONS FOR WEARING SOFT AND HARD SHELL HELMETS**

The helmet should be worn as prescribed by the doctor.

### ***Care of the Helmet***

Once a day wash the helmet with mild soap or shampoo. Rinse thoroughly with water. If the individual wearing the helmet has a tendency to have concerns with their skin, wipe the interior surfaces of the helmet with 70% alcohol to destroy bacteria and to remove soap residues that may cause the irritation. Allow to dry well.

### **Important**

If at any time you are experiencing problems or think you may have a problem, call immediately for an appointment if there are any skin problems, discomfort or other fitting problems.

### **Caution**

When not wearing the helmet, keep it away from pets since they are attracted to the human scent and will chew and destroy it.

Warranty: The components of this device are warranted for 90 days, after which a service charge, based on an hourly rate and materials, can be applied. The warranty does not apply to anatomical changes of the patient, misuse and/or loss of the device or any alterations made by anyone other than Great Lakes.