

**GREAT LAKES ORTHOPEDIC LABS, INC.**

2362 Genesee Street, Cheektowaga, NY 14225

Telephone. No. (716) 893-4116

**GREAT LAKES ORTHOPEDIC LABS, INC.**

219 Bryant Street, Buffalo, NY 14222

Telephone No. (716) 878-7307

**DIRECTIONS FOR WEARING**  
**UCBL STYLE ARCH SUPPORT**

- 1) Always wear socks when wearing the UCBL.
- 2) Fit the UCBL on the foot making sure the heel is all the way back into the heel section.
- 3) Wear sneakers over the UCBL and lace up the sneakers snugly or place the UCBL into the sneaker first, then step into the sneaker and lace it up snugly.
- 4) Wash the UCBL with soap and water using soap such as ivory or other soaps safe for skin use.
- 5) On initial wearing of the UCBL, check after one hour for pressure spots. Gradually increase the wearing time and periodically check the skin for pressure spots. All red marks should disappear within twenty (20) minutes once the UCBL is off. If red marks persist for longer than twenty (20) minutes, call our office for an appointment to have the UCBL adjusted. If you need immediate assistance after regular office hours, please call 893-4116 and leave a message on the answering machine. Your call will be returned.
- 6) Caution: When not wearing the device, keep it away from your pets since they are attracted to the human scent and will chew and destroy it.

Warranty: The components of this device are warranted for 90 days, after which a service charge, based on an hourly rate and materials, can be applied. The warranty does not apply to anatomical changes of the patient, misuse and/or loss of the device or any alterations made by anyone other than Great Lakes.